WITH SAFETAC® TECHNOLOGY

## GENTLE SECUREMENT TAPE

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Distributed by:  
CVS Pharmacy, Inc.  
One CVS Drive, Woonsocket, RI 02895  
CVS.com® 1-800-SHOP CVS

Master PD-508091 Rev. 01

Manufactured by:  
Mölnlycke Health Care AB  
Gamlestadsvägen 3C, Box 13080, SE-402 52 Göteborg, Sweden

**40438-00**  
512455 Rev. 01  
Made in Finland

### What is Mepitac®?

Mepitac is a soft silicone tape that is water resistant and can be lifted and reapplied.

It features a proprietary technology known as Safetac.

### About Safetac® Technology

Safetac is a unique adhesive technology, with the following benefits:

- Provides less pain upon removal
- Sticks gently to the skin
- Skin friendly, appropriate for sensitive skin

You can find Safetac technology in the following wound care brands: Mepilex® Border Lite, Mepilex® Lite, Mepitel® Film, Mepiform® and Mepitac®. Read more at [www.molnlycke.com](http://www.molnlycke.com)

### What is Mepitac® for?

Mepitac gently but yet securely holds tubing, wound and surgical dressings, and it can be used for gentle protection of fragile skin.

### Precautions

Do not use if allergic or sensitive to silicone, or plastics including polyurethane, polyamide, polyester or polyethylene.

### Other information

Mepitac is a single patient use product. Never reuse!

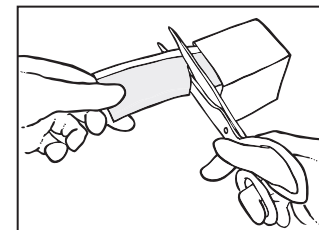
Store in dry conditions and do not refrigerate.

Excessive sweating may cause Mepitac to not stick well.

Mepilex, Mepitel, Mepiform, Mepitac and Safetac are registered trademarks of Mölnlycke Health Care AB.


Patent:  
US 6,479,724

### Instructions for use

Make sure the skin is clean and dry and free from any ointment, lotion, etc, because Safetac sticks best to clean, dry skin.

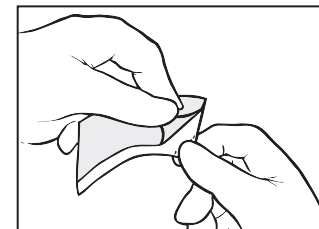Cut off the desired length of Mepitac using scissors.

Remove the protective release film and apply Mepitac without stretching.

Mepitac can be lifted and re-applied during application.

Issued 2016-05

WITH SAFETAC® TECHNOLOGY

## GENTLE SECUREMENT TAPE

This product and its packaging are not made with natural rubber latex

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Mepilex, Mepitel, Mepiform, Mepitac y Safetac son marcas registradas de Mölnlycke Health Care AB.