

CAD Ref: PLSM129

Dimensions: 123 x 27 x 73

Mölnlycke®

Ref no: 47642-01
Created by: Natalie Whiteside

Refer to ARS PD-530092 and Master Text IFU PD-572948

Mölnlycke Health Care, Gamlestadsvägen 3C, SE-402 52, Göteborg.

Colour information

- PMS 361
- PMS Cool Grey 10
- Cutter
- Varnish free

Mölnlycke®

Natural 0.5m

47642-01

Mölnlycke®

Tubigrip®

Elasticated tubular support bandage

Natural 0.5m

See instructions for use

LATEX Contains natural rubber latex

25°C Upper temperature limit
77°F

Keep dry



CE



Natural 0.5m

Mölnlycke®

Tubigrip®

Elasticated tubular support bandage






Provides firm support for

- Sprains • Strains • Swelling

See reverse for size guide

Natural 0.5m

Made in UK

www.molnlycke.com


The Mölnlycke and Tubigrip trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care Group of Companies.

SE-402 52 Göteborg, Sweden

Mölnlycke Health Care AB
Gamlestadsvägen 3C, Box 13080

Manufacturer

SIZE	S	M	L	M/L	L	M	S
B	S	M	L	M/L	L	M	S
C	S	M	L	M/L	L	M	S
D	S	M	L	M/L	L	M	S
E	S	M	L	M/L	L	M	S
F	S	M	L	M/L	L	M	S
G	S	M	L	M/L	L	M	S

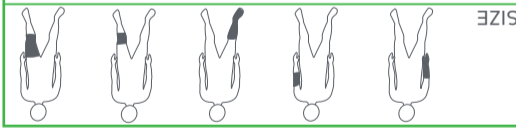

Elasticated Tubular Bandage BP Type A

- Easy to apply • Stays in place

Provides firm support for sprains, strains and swelling in adults, adolescents and children.

Instructions for use: Consult size guide. Cut the bandage to twice the length of the affected limb, if required make 2 slits for a thumb hole. Pull over the limb, then double back. Disposal should be handled in accordance with local environmental procedures.

Precautions: Do not use Tubigrip if known hypersensitivity to the fibre content. If your condition worsens stop use and consult a health care professional. Keep clean, if soiled discard and replace bandage. If any serious incident has occurred in relation to the use of Tubigrip, it should be reported to Mölnlycke Health Care.

Fibre Content: Cotton, Elastodiene and Polyester

Caution: This product contains natural rubber latex which may cause allergic reactions, including anaphylactic responses.

Mölnlycke®

0.5m